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## EN Directions for use of the Stericlin® Bowie & Dick Helix Test

**The stericlin® Bowie & Dick Helix Test has been designed for the performance test in steam sterilizers. It consists of two parts: the PCD (Process Challenge Device) and the indicator strip which is specifically tailored to the needs of this PCD. The pigments for the indicator strip do not contain any metals. The indicator system is odorless and does not leave color stains. To ensure immaculate performance of this highly sensitive indicator, the below directions for use should be carefully read and acted upon.**

In combination with the single-use Indicator System, the PCD is designed for a maximum of 250 Bowie & Dick applications \*1. Thus each PCD is supplied with 250 indicator strips. The serial number (SN) of the PCD and the container of the supplied indicator strips are identical. Any PCD with individual SN is hence to be used with indicator strips carrying the same SN and LOT respectively.

**Warm up the sterilizer with an empty chamber run before starting the test-run.**

**Preparation of the B&D Helix Test**

Fold the indicator strip so that the labelling and the indicator field face inwards. Insert the indicator strip into the plastic capsule in the direction of the arrows. The arrows are to remain visible and point upwards. Screw the capsule with the strip into the process challenge device as far as it will go (a spare capsule is provided with each unit).

**B & D test cycle and assessment**

Place the stericlin® Bowie & Dick Helix Test in an upright position inside the sterilizer, e. g. on the lowest level of the chamber, above the drain of the empty and preheated chamber, or near the door of the chamber. Start the Bowie & Dick test-run (at 134 °C for 3.5 minutes \*2). After finishing the complete test run, immediately remove the Helix Test from the chamber and open it. Examine the indicator strip and document the result.

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**IMPORTANT NOTE:** Beware of burns - upon sterilization the PCD is very hot.

**Evaluation**

To examine the results of the test run, put the autoclaved indicator strip on a bright surface.

**Please note:** Evaluation of the results of the autoclaved indicator strip **held against a source of light is not appropriate**. Analyze the results by comparing the shade of the indicator with the shades displayed on the indicator reference chart. If the sterilizer has accomplished proper steam penetration with sufficient steam quality, the color shade of the indicator will have completely and uniformly changed from bright blue to pink.

If no sufficient steam penetration or minor steam quality has been accomplished, the indicator's shade will not have changed completely. **In this case, please check your sterilizer.** Result documentation is legally sound and sufficient with signature and countersignature (supervisor).

\*1) After 250 applications the entire PCD can be disposed of as metal scrap without any concern.
\*2) Temperature and time can slightly vary according common tolerances; please see e. g. EN 285 and EN 13060

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